# OTORIG<sup>®</sup>

## ORAL SPRAY

### Shock therapy of the oro-pharyngeal mucosa

with

OTORIG<sup>®</sup> ORAL SPRAY with Ozoile<sup>®</sup> is a medical device for prevention and treatment of oro-pharyngeal mucosa affections both acute and chronic in case of irritation, swelling, burning, itching, difficulty in swallowing, oxidative stress with different etiology. Innovative formulation, creates a protective lipidic film hostile to microbiota alterations which protects and restores the good condition.

#### MAIN COMPONENTS:

Ozoile<sup>®</sup> (Stable Ozonides), Aqua (Ozonized), Menthol, Sorbitol, Glycerin.

#### OZOILE<sup>®</sup>

Acts on the irritative and itchy state, brings oxygen on site and creates an unfavorable environment to microbiota alterations. Contrasts redness, itching, burning, swelling, desquamation and degenerative processes in progress.

#### MENTHOL

Refreshing analgesic action, gives a relief sensation with expectorant and antitussive properties.

#### SORBITOL

Natural sweetener, stimulates salivation lubricating the mucosa.

#### GLYCERIN

Moisturizing of the mucosa, it has emollient and protective properties against peripheral receptors that trigger the cough reflex.

#### INSTRUCTIONS

Otorig<sup>®</sup> oral spray with Ozoile<sup>®</sup> is a medical device, specific for prevention and treatment of both acute and chronic oro-pharyngeal mucosa affections. Creates a protective lipidic film against external aggressions and environmental adverse

factors, creating an unfavourable environment to microbiota alterations. Otorig<sup>®</sup> oral spray is indicated for all those situations involving the mouth, gums, throat, characterized by severe irritation, swelling, burning, itching, difficulty in swallowing, oxidative stress with different etiology, adjuvant in case of hoarseness and cough of various kinds, influenzal syndromes or from cooling, after-effects of ENT surgery. Innovative formulation with Ozoile<sup>®</sup>, Otorig<sup>®</sup> oral spray creates a protective barrier that gives immediate relief, preserves the microbiota and restores the good condition of the mucosa of the oral mucosa.

#### HOW TO USE

Otorig<sup>®</sup> oral Spray is equipped with two interchangeable dispensers:

- classic spray dispenser whose spray covers a larger surface;
- throat spray dispenser for a targeted spray.

Spray the product directly on the affected area, 3-4 puffs, 4-5 times a day or, as appropriate, as directed by physician. The period of treatment differs according to the rapidity of the therapeutic response.

#### WARNINGS AND PRECAUTIONS

Keep out of the reach of children. Do not use the device beyond the expiration date. Do not use it if the packaging appears to be damaged. Avoid the contact with eyes, in case of accidental contact rinse thoroughly with water. There are no cases of sensitization or side effects, if this phenomenon occurs please discontinue use and consult your physician. Wash your hands well before using the medical device. Carefully close the bottle after use. Store in a cool, dry place, away from heat sources and direct sunlight. The expiration date refers to the intact and correctly stored product. After opening it is recommended to use the product within 6 months.

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