# RIGENOMA®

## BARRIER CREAM

### Topical innovation Protects and Repairs the Skin

with

RIGENOMA® BARRIER CREAM with Ozoile® and Cera Alba protects and repairs the skin in case of dermatitis associated with incontinence, macerated erythema localized in different body areas (axillary, mammary, inguinal, etc.), eczematous lesions and impetiginization, prevents pressure injuries and acts effectively and quickly against the signs and symptoms that characterize these ailments: irritation, burning, itching, desquamation, de-epithelialization. The combination of Ozoile® and Cera Alba into an innovative formulation creates a protective barrier film with a non-occlusive shielding effect that promotes the physiological repair processes of the skin and creates an unfavorable environment to microbial proliferation. Rigenoma® barrier cream allows the adhesion with other dressings.

#### MAIN COMPONENTS:

Ozoile®(Stable Ozonides with Vitamin E acetate), Cera Alba, Ribes Nigrum Leaf extract, Sodium Hyaluronate, Glycyrrhetic Acid, Melaleuca Alternifolia Leaf Oil.

#### OZOILE®

Acts on the inflammatory and itchy state, counteracts the oxidative stress and creates an unfavorable environment to the microbiota alterations. Effective against redness, itching, burning, swelling, desquamation and degenerative processes in progress. It brings oxygen in situ and promotes the re-epithelialization and healing, contrasts radiation damages.

#### CERA ALBA

Film-forming agent, creates a protective barrier with a non-occlusive shielding effect, promotes tissue repair.

#### SODIUM HYALURONATE

Contributes to the physical protection of the skin for its visco-elastic properties. It promotes hydration and re-epithelialization.

#### RIBES NIGRUM

Deredness action.

#### GLYCYRRHETINIC ACID

Acts on irritation, burning, itching.

#### MELALEUCA ALTERNIFOLIA LEAF OIL

Enhances the action of Ozoile®.

#### INDICATIONS

Rigenoma® barrier cream is indicated in the prevention and treatment of skin lesions from incontinence and maceration, even with overlapped bacterial colonization (impetiginization) and/or fungal infections and in the prevention of pressure sores. Its use is specific in case of dermatitis associated with incontinence, macerated erythema localized in different body areas (mammary, axillary, inguinal, etc.), eczematous lesions and impetiginization. Rigenoma® barrier cream with Ozoile® protects against adhesive and rubbing trauma, allows the adhesion with other dressings.

#### HOW TO USE

After cleansing and after drying the skin, apply a thin layer of Rigenoma® barrier cream evenly, once or twice a day or as directed by physician. In case of incontinence Rigenoma® barrier cream must be applied at each change of the diaper. The treatment period differs to the subjectivity of the therapeutic response.

#### WARNINGS

For external use. Do not ingest. Keep out of the reach of children. Do not use it if the packaging appears to be damaged. Do not use this medical device beyond the expiration date. The expiration date refers to the intact and correctly stored product. In case of eye contact rinse thoroughly with water. Store in a cool, dry place, away from heat sources and direct sunlight. Store preferably between 5° C and 35°C. Wash your hands well before using the medical device. Carefully close the tube after use. After opening it is recommended to use the product within 6 months. Rigenoma® barrier cream is well tolerated. There are no cases of sensitization or side effects. Do not use in case of ascertained hypersensitivity to the product or some of its components; if this phenomenon occurs please discontinue use and consult your physician. There are no known contraindications and secondary effects in case of pregnancy and feeding time; in any case it is recommended to consult your physician before the application.

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