RIGENOMA®

HYDROGEL

Topical innovation in the treatment of Necrotic and Poorly Cleaned Skin Lesions

with

RIGENOMA® HYDROGEL with Ozoile® is an innovative water-based gel formulation, pH 4.2, which promotes the natural cleaning / debridement of the wound also infected by autolysis, thanks to hyperhydration of the necrotic tissue. It softens and absorbs slough and exudate and promotes at the same time the granulation of the vital tissue below. Its particular formulation with Ozoile® and Lactic acid inserted in an aqueous gel ensures a humid wound environment, facilitates the autolytic debridement and the cleansing of the wound bottom covered by necrosis, fibrin and cellular debris, creates an unfavorable environment to microbial proliferation and promotes the recovery tissue repair.

MAIN COMPONENTS

Aqua (Ozonized), Ozoile® (Stable Ozonides), Lactic Acid.

INDICATIONS

Treatment of necrotic and poorly cleaned skin lesions. Under medical supervision, it can also be used on infected wounds.

HOW TO USE

After the removing of the secondary dressing and the cleansing of the lesion, apply a layer of

Rigenoma® hydrogel suitable to cover entirely wound. Apply the product twice or three times a week. The dressing change must be commensurate with the exudation of the wound. Protect with secondary dressing.

WARNINGS

For external use. Do not ingest. Keep out of the reach of children. Do not use it if the packaging appears to be damaged. Do not use this medical device beyond the expiration date. The expiration date refers to the intact and correctly stored product. In case of eye contact rinse thoroughly for several minutes. Remove any contact lenses if it is easy to do. Continue rinsing. Store in a cool, dry place, away from heat sources and direct sunlight. Store preferably between 5° C and 35°C. Wash your hands well before using the medical device. Carefully close the tube after use. After opening it is recommended to use the product within 6 months. Rigenoma® Hydrogel is well tolerated. There are no cases of sensitization or side effects. Do not use in case of ascertained hypersensitivity to the product or some of its components; if this phenomenon occurs please discontinue use and consult your physician. There are no known contraindications and secondary effects in case of pregnancy and feeding time; in any case it is recommended to consult your physician before the application.

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PAKIRANJE

Tuba s 100 ml.

BREZ GLUTENA
BREZ BARVIL
BREZ ALERGENOV

 **Distributor:**
MagPharm,
prodaja farmacevtskih izdelkov, d.o.o.
 Litostrojska 44e,
1000 LJUBLJANA

 **ERBAGIL s.r.l.**
Via L. Settembrini, 13
82037 Telese Terme (BN) Italy
www.erbagil.com

REV. 00 02/2024