# RIGENOMA®

## SPRAY

**The shock therapy for injured,  
inflamed skin to sanitize**

with

### INSTRUCTIONS

RIGENOMA® SPRAY with Ozoile® is indicated for prevention and treatment of acute and chronic skin lesions with different etiology even if there is a risk of infection such as bedsores, vascular, metabolic, pressure ulcers, surgical wounds, traumatic, chemo and radiotherapy injuries. The product is also indicated in prevention and treatment of dermatological pathologies characterized by inflammation, irritation, erythema, xerosis, itching, desquamation, alterations of flora and skin barrier, possible infection as in the case of mycosis.

### MAIN COMPONENTS:

Ozoile® (Stable Ozonides with Vitamin E acetate), Melaleuca Alternifolia Leaf Oil, Thioctic Acid.

Ozoile®, stable Ozonides with Vitamin E acetate, is an innovative molecule pool obtained from a patented process with anti-inflammatory and anti-itching action, contrasts hypoxia, creates an unfavorable environment to pathogenic microbial proliferation and promotes tissue repair processes.

### HOW TO USE

Spray the product directly on the affected area. In the treatment of acute and chronic lesions with different etiology, after cleansing with Rigenoma® detergent, apply Rigenoma® spray on the bottom of the lesion, one spray every 4 cm<sup>2</sup> approximately, at a distance of approximately 5 cm. To complete the dressing, apply a homogeneous layer of Rigenoma®

cream on periwound skin and spread a veil on the surrounding region for more 5 cm, finally cover the lesion with Rigenoma® gauze to protect, keep the environment humid, promote tissue repair and regeneration actively dialoguing with the wound bottom. Apply Rigenoma® spray once or twice a day or in case of bandage and occlusive dressing, every 3 or 4 days to change the dressing as indicated by the physician. In the treatment of dermatological problems, apply Rigenoma® spray 2 or 3 times a day. The treatment period differs according to the treated area and the rapidity of the therapeutic response.

### WARNINGS AND PRECAUTIONS

Topical use. Do not ingest. Keep out of the reach of children. Do not use it if the packaging appears to be damaged. Do not use the device beyond the expiration date. The expiration date refers to the intact and correctly stored product. Store in a cool, dry place, away from heat sources and direct sunlight. Store preferably between 5° C and 35°C. Wash your hands well before using the medical device. Carefully close the tube after use. After opening it is recommended to use the product within 6 months.

### CONTRAINDICATIONS

Rigenoma® spray is well tolerated. It is contraindicated in case of ascertained hypersensitivity towards the components. There are no cases of sensitization or side effects, if this phenomenon occurs please discontinue use and consult your physician.

There are no known overdoses cases. There are no known interactions with drugs and medicinal substances.

### PACKAGE

Bottle 20 ml - 0.7 fl. Oz.

GLUTEN FREE  
NO ALLERGENS  
NO COLOURS  
NO PRESERVATIVES

 **ERBAGIL s.r.l.**  
Via L. Settembrini, 13  
82037 Telese Terme (BN) Italy  
[www.erbagil.com](http://www.erbagil.com)

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