OTORIG[®]

NASAL SPRAY

Shock therapy of nasal mucosa

with

OTORIG[®] NASAL SPRAY is a protective, decongestant and sanitizing medical device of nasal cavities specific for prevention and treatment of nasopharyngeal affections both acute and chronic in case of irritation, swelling, itching, burning, excessive mucus production. Creates a protective mucoadhesive lipidic film hostile to microbiota alterations and restores the good condition of the nasal mucosa.

MAIN COMPONENTS

Ozoile[®](Stable Ozonides), Aqua (Ozonized), Menthol, Sodium Chloride.

OZOILE[®]

It acts on the irritative and itchy state, brings oxygen on site and creates an unfavorable environment to microbiota alterations. Contrasts redness, itching, burning, swelling, desquamation and degenerative processes in progress.

MENTHOL

Refreshing analgesic action, gives a relief sensation.

SODIUM CHLORIDE

Its presence in solution promotes the attraction of osmosis water from the congested mucosa and it fluidifies the mucus, facilitating its removal.


INSTRUCTIONS

Otorig[®] nasal spray with Ozoile[®] is a protective, decongestant and sanitizing medical device of nasal cavities, specific for prevention and treatment of both acute and chronic nasopharyngeal affections characterized by irritation, swelling, itching, burning, excessive mucus production. Innovative formula, it is a valid help in case of flu syndromes, caused by cooling, allergic or environmental pollutants and surgical after-effects in ORL. Otorig[®] nasal spray decongests the nasal passages, reduces the obstruction and regularizes secretions by promoting an osmotic exchange which favors the attraction of fluids from the mucosa and it fluidifies the mucus, facilitating its removal. Its formulation with Ozoile[®] keeps the mucosa well hydrated and it has an important sanitizing and protective action creating a mucoadhesive lipidic film hostile to microbiota alterations, that restores the good condition of the nasal cavities.


HOW TO USE

1) Remove the transparent protection cap (a) and the classic spray dispenser (b) by pulling upwards.

2) Place the nasal spray dispenser on the plunger (c).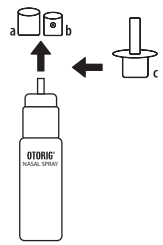

Apply Otorig[®] Nasal Spray as needed 1 or 2 puffs per nostril, blow the nose gently, then repeat the application.

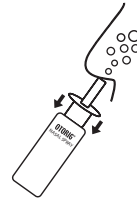It is recommended to use the product more times during the day even at close range. Before going to bed, repeat the dispensing. Rinse the dispenser at the end of each use by positioning it under a jet of hot water, then cover the dispenser with the protective cap. Since it is a natural product, thanks to its no pharmacological mechanism of action, it is not addictive and can be used frequently and for prolonged periods.

WARNINGS AND PRECAUTIONS

Keep out of the reach of children. Do not use the device beyond the expiration date. Do not use it if the packaging appears to be damaged. Avoid the contact with eyes, in case of accidental contact rinse thoroughly with water.

There are no cases of sensitization or side effects, if this phenomenon occurs please discontinue use and consult your physician. Wash your hands well before using the medical device. Carefully close the bottle after use. Store in a cool, dry place, away from heat sources and direct sunlight. The expiration date refers to the intact and correctly stored product. After opening it is recommended to use the product within 6 months.

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