# DERMOZOIL®

## CREAM

### Repairs and Regenerates the Skin

with

### INDICATIONS

DERMOZOIL® CREAM is indicated in the prevention and treatment of Dermatitis and Dermatoses with different etiology such as Atopic Dermatitis, Eczematous Dermatitis and dermatological pathologies characterized by inflammation, irritation, erythema, xerosis, itching, flaking, possible infection, alterations of the flora and skin barrier, such as Acne and Psoriasis. Dermozoil® Cream is also indicated for prevention and treatment of acute and chronic skin lesions with different etiology.

### MAIN COMPONENTS

Ozoile® (Ozonides stable with Vitamin E acetate), Ribes nigrum Leaf Extract, Sodium Hyaluronate, Calendula officinalis Flower Extract, Equisetum Arvense Extract, Ruscus Aculeatus Root Extract, Thioctic Acid.

Ozoile®, Stable ozonides with Vitamin E acetate, is a pool of innovative molecules obtained from a patented process that has anti-inflammatory and anti-itching action, counteracts hypoxia, creates an unfavourable environment to pathogenic microbial proliferation and promotes tissue repair processes.

### HOW TO USE

Spread the product directly on the affected area. Apply Dermozoil® Cream in the treatment of dermatological pathologies two or three times a day. In the treatment of acute and chronic skin lesions with different etiology, the product can also be applied directly to the entire injured area especially during the granulation phase,

on the peri-wound skin and to the surrounding area for more than 5 cm. The treatment period differs according to the treated area and the rapidity of the therapeutic response.

### WARNINGS AND PRECAUTIONS

Topical use. Do not ingest. Keep out of reach of children. Do not use it if the packaging appears to be damaged. Do not use the device beyond the expiration date. The expiration date refers to the intact and correctly stored product. Store in a cool, dry place away from heat sources and direct sunlight. Store preferably between 5 °C and 35 °C. Wash your hands well before using the medical device. Carefully close the tube after use. After opening it is recommended to use the product within 6 months.

### CONTRAINDICATIONS

Dermozoil® Cream is well tolerated. It is contraindicated in case of ascertained hypersensitivity towards the components. There are no cases of sensitization or side effects, if this phenomenon occurs please discontinue use and consult your physician. There are no known overdoses cases. There are no known interactions with drugs and medicinal substances.

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### PAKIRANJE

100 ml tuba.

BREZ GLUTENA  
BREZ BARVIL  
BREZ ALERGENOV

Distributor:  
MagPharm,  
prodaja farmacevtskih izdelkov, d.o.o.  
Litostrojska 44e,  
1000 LJUBLJANA

ERBAGIL s.r.l.  
Via L. Settembrini, 13  
82037 Telese Terme (BN) Italy  
[www.erbagil.com](http://www.erbagil.com)

REV. 02 01/2023