# OTORIG<sup>®</sup>

## DROPS

### For treatment and cleansing of irritated, itchy and flaky external auditory canal

with

**OZOILE<sup>®</sup>**  
ERBAGIL PATENT  
Stable Ozonides

OTORIG<sup>®</sup> DROPS with Ozoile<sup>®</sup> creates a protective lipid film against external aggressions that restores the good condition of the skin and creates an unfavorable environment to the local microbiota alterations. Effective against irritation, flaking, itching, burning, swelling, counteracts oxidative stress and promotes the correct re-epithelialization.

#### MAIN COMPONENTS

Ozoile<sup>®</sup> (Stable Ozonides with Vitamin E acetate), Menthol, Melaleuca Alternifolia Leaf Oil.

#### OZOILE<sup>®</sup>

Acts against the irritative and itchy status and creates an unfavorable environment to the microbiota alterations. It acts against redness, itching, burning, swelling, flaking and degenerative processes in progress. It transfers oxygen to the tissue and enhances the re-epithelialization.

#### VITAMINE E ACETATE

Soothing effect, it contrasts membrane lipid peroxidation.

#### MALALEUCA ALTERNIFOLIA

Enhances the action of Ozoile<sup>®</sup>.

#### MENTHOL

Refreshing analgesic action gives a relief sensation.

#### INDICATIONS

For the specific cleansing and treatment of external auditory canal in case of irritation, itching, burning, swelling, desquamation, degenerative process in progress.

It protects the ear from adverse environmental factors (dust, pollution, humidity) it promotes the re-epithelialization of

the external ear canal. It refreshes and releases the ear canal from excess earwax, prevents and contrasts the making of plug of earwax.

#### HOW TO USE

After removing the sleever and the upper cap, instill with the special dropper in the ear canal keeping your head tilted to one side.

Adults: 3-4 drops twice a day.

Children: 1-2 drops twice a day.

The treatment period differs according to the treated area and the rapidity of the therapeutic response.

#### WARNINGS

Do not ingest and keep out of reach of children. Do not use this medical device beyond the expiration date. Do not use it if the packaging appears to be damaged. The product becomes viscous at low temperatures. To make the flow easier and draw the drops out heat up the product by rubbing the bottles with both hands or by using others heat sources. Do not exceed a temperature of 20°C. There are no cases of sensitization or side effects, if this phenomenon occurs, it is recommended to discontinue the treatment and consult your physician. Wash your hands well before use of medical device. Firmly close bottle after use. Keep the product in a dry place far from heat source and from sun light exposure. The expiration date is referred to the product intact and perfectly preserved. After opening it is recommended to use the product within 6 months.

#### PACKAGE

10 ml bottle - 0.3 fl. Oz.

GLUTEN FREE  
NO COLOURS  
NO ALLERGENS  
WITHOUT PRESERVATIVES

 **ERBAGIL s.r.l**  
Via L. Settembrini, 13  
82037 Telese Terme (BN) Italy  
[www.erbagil.com](http://www.erbagil.com)

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