# RIGENOMA®

## ALGINAT

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# RIGENOMA®

## ALGINATE

### Adsorbent dressing in alginate soaked in Ozoile® for the treatment of skin lesions

with

RIGENOMA® ALGINATE is a sterile dressing for exuding wounds composed of natural calcium alginate fibers in combination with Ozoile® which promotes the formation of granulation tissue and creates an unfavorable environment to microbial proliferation. Rigenoma® alginate is packaged in a protective blister and sealed inside a sterile box. The final product is beta rays sterilized. The particular weave of the alginate fibers ensures excellent absorbency and resistance to fluids that allows atraumatic removal, simple and in a single gesture, without leaving residues. The gel formed by alginate ensures excellent control of exudate with the absence of macerative phenomena on the periwound skin and the re-epithelialization border. The combination with Ozoile® promotes the formation of granulation tissue and creates an unfavorable environment to microbial proliferation.

#### MAIN COMPONENTS

Calcium Alginate, Ozoile® (Stable Ozonides).

#### INDICATIONS

Rigenoma® alginate is a sterile dressing indicated for the local treatment of acute and chronic lesions with different etiology, from moderately to highly exuding, superficial and cavitory, such as surgical, traumatic wounds, vascular injuries, pressure, diabetic foot, oncological and induced by radiotherapy lesions, etc. Rigenoma® alginate is a dressing that promotes primary homeostasis and can control minor bleeding in superficial and cavitory lesions.

Rigenoma® alginate is indicated in the treatment of infected wounds under medical supervision.

#### HOW TO USE

After cleansing the lesion and after drying its edges with a sterile gauze, apply Rigenoma® alginate directly on the bottom of the wound, both on flat lesions avoiding excessive overlap at the edges, and in cavitory lesions to fill the loss of substance. Thanks to the formation of gel, Rigenoma® alginate adapts to the walls of the cavitory lesion. Cover with a non-adherent secondary dressing such as Rigenoma® gauze and secure with a containment bandage. The Rigenoma® alginate dressing should be replaced two or three times a week, based on the exudate quantity, when it is saturated and its gelling complete.

#### WARNINGS

For external use. Do not ingest. Keep out of the reach of children. Do not use it if the packaging appears to be damaged. Do not use this medical device beyond the expiration date. The expiration date refers to the intact and correctly stored product. Store in a cool, dry place, away from heat sources and direct sunlight. Store preferably between 5°C and 35°C. Wash your hands well before using the medical device. The sterile single packs are for single use, this means that the gauze can be used only once and by a single patient. Reuse it can cause serious infections.

#### SIDE EFFECTS

Rigenoma® alginate is well tolerated. There are no known cases of sensitization and side effects to date, do not use in case of ascertained hypersensitivity to the product or some of its components; if these phenomena occur please discontinue use and consult your physician. There are no known contraindications and secondary effects in case of pregnancy and feeding time; in any case it is recommended to consult your physician before the application.

#### PACKAGE

5 dressing 10 cm x 10 cm.

NO ALLERGENS

NO COLOURS

GLUTEN FREE

 **ERBAGIL s.r.l.**  
Via L. Settembrini, 13  
82037 Telesse Terme (BN) Italy  
[www.erbagil.com](http://www.erbagil.com)

REV. 01 01/2023