RIGENOMA®

GAZA

IMPREGNIRANE GAZE ZA PREPREČEVANJE IN ZDRAVLJENJE LEZIJ

5
sestavino

OZOILE®
ERBAGIL PATENT
Stabilni ozonidi

INDIKACIJE

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RIGENOMA®

GAUZE

IMPREGNATED GAUZES FOR THE PREVENTION AND TREATMENT OF LESIONS

with

INDICATIONS

RIGENOMA® GAUZE is an advanced medicament dressing indicated for the prevention and treatment of acute and chronic skin lesions, even in case of lesions at risk of infection such as, bedsores, vascular, metabolic ulcers, from pressure, surgical wounds and actinic ulcers.

MAIN COMPONENTS

Ozoile® (Stable Ozonides with Vitamin E acetate), Sodium Hyaluronate, Aloe Barbadensis Leaf Juice, Sedum Purpureum Extract, Ribes nigrum Leaf Extract, Glycyrrhetic Acid, Ruscus Aculeatus Root Extract, Calendula officinalis Flower Extract, Thiocetic Acid.

HOW TO USE

Open the bag, remove the protective plastic film, remove the gauze, apply one or more depending on the extension of the area to be treated, cover everything with sterile gauze or appropriate bandage as needed.

Repeat the medication once day or every other day depending on the situation and doctor instructions. In case of bandaging or occlusive dressing the product can be replaced every three or four days as per physician instructions.

The treatment period differs according to the area to be treated and the rapidity of the therapeutic response.

WARNINGS AND PRECAUTIONS

Topical use. Do not ingest. Keep the product out of reach of children. Do not use in case of damaged packaging. Do not use beyond the expiration date. The expiration date refers to the intact and correctly stored product. Store in a dry place away from heat and direct sunlight.

The gauze is disposable; this means that it can only be used once and by a single patient only. Re-use can cause serious infections.

CONTRAINDICATIONS

Rigenoma® gauze is well tolerated. To date, there are no known cases of sensitization and side effects, if this phenomenon should occur discontinue use and consult immediately your physician.

SIDE EFFECTS

There are no know interactions or incompatibilities.

PACKAGE

Box with 10 gauzes.

 **ERBAGIL s.r.l.**
Via L. Settembrini, 13
82037 Telese Terme (BN) Italy
www.erbagil.com

REV. 03 02/2024